

SD BIOLINE

HIV/SYPHILIS DUO

PRODUCT DESCRIPTION

SD BIOLINE HIV/Syphilis Duo is a rapid, qualitative test for the detection of antibodies to all isotypes (IgG, IgM and IgA) specific to HIV-1/2 and *Treponema pallidum* (TP) simultaneously in human serum, plasma or whole blood.

KIT CONTENTS

06FK30: 1 kit contains 25 test devices, 1 assay diluent and 1 instructions for use.

06FK35: 1 kit contains 25 test devices, 1 assay diluent and 1 instructions for use, 25 alcohol swabs, 25 lancets, 25 capillary pipettes.

PERFORMANCE

	HIV	SYPHILIS
Sensitivity	99.91%	99.67%
Specificity	99.67%	99.72%

* Internal evaluation of SD BIOLINE HIV/Syphilis Duo test

TYPE OF SPECIMEN

Whole blood, serum, plasma

RUNNING TIME FOR TEST RESULT INTERPRETATION

15-20 minutes

PACKAGING INFORMATION

- Dimensions: 152 x 125 x 71 mm (25 tests)
- Volume : 1,349 cm³ (25 tests)
- Weight :
 - 250 g (06FK30)
 - 260 g (06FK35)

SHIPPING CARTON

- 25 tests per kit, 60 kits per carton
- Dimensions : 515 x 475 x 390 mm
- Volume : 95,404 cm³
- Weight : 60 kits
 - 15.06 kgs (06FK30)
 - 15.77 kgs (06FK35)

ABBOTT.COM/POCT

1. Sensitivity and Specificity are extracted from the Instructions for use of each product.

© 2019 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. 120002054-03, 05/19

STORAGE CONDITIONS

- Storage temperature : 1-30 °C.
- Keep away from extreme heat and humidity.
- Do not freeze test kits or reagent.
- Full effectiveness of expiry date is under ideal storage conditions.
- Stability of kit components is guaranteed until expiry date.
- DO NOT use any components after expiry date.

SHELF LIFE

24 months

QUALITY CONTROL

A procedural control is included in the test device as a control line. If the control line does not appear after 20 minutes, the test is invalid and the specimen should be tested with a new device.

MATERIALS REQUIRED BUT NOT PROVIDED

- A pair of disposable gloves
- Timer
- Pen
- A biohazard box
- A non-sharps disposal bin

