



SD BIOLINE hCG, hCG(U/S)

One Step Human chorionic gonadotropin Test

Human chorionic gonadotropin (hCG) is a glycoprotein hormone secreted by the developing placenta shortly after fertilization. When you are pregnant, your body will produce special pregnancy hormone known as hCG.



General Information

SD BIOLINE hCG test is an immunochromatographic assay designed for qualitative detection of hCG in urine.

	hCG	hCG(U/S)
Test result	3 minutes	5 minutes
Detection	25 mIU/ml	25 mIU/ml
Specimen	Urine	Urine, Serum
Storage temperature	2~30 °C	1~30 °C

- Cross reactivity : No cross reactivity with 200 mIU/ml of hLH, 1,000 mIU/ml of hFSH and 1,000µIU/ml of hTSH


Materials Provided

- Test device / Strip
- Disposable dropper


Test Procedure

1 Add Specimen

Device
Dispense
**3-4 drops of urine
or serum into the
sample well "S".**

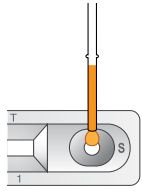

Strip
**Insert strip into
the test tube.**

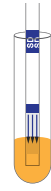

Urine
Wait 3 mins.




OR

Urine, Serum
Wait 5 mins.



2 Read Results

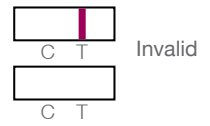

Ordering Information

Product	Cat. No.	Type	Specimen	Pack Size
hCG	30FK10	Device	Urine	25T/Kit
hCG Fast	30FK12	Strip	Urine	25Tx4/Kit
hCG(U/S)	30FK20	Device	Urine, Serum	25T/Kit