


IVD CE



Adeno Ag

# Capilia™ Adeno Neo

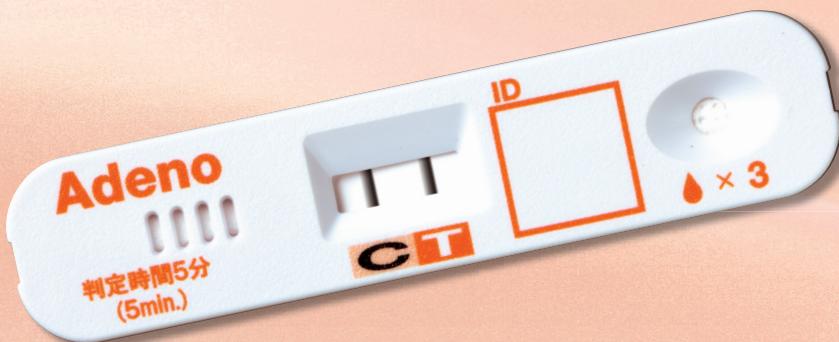

\* The sample shows positive result.

■ **Rapid test for detecting Adenovirus antigen**  
Permitting accurate results; simple and easy to use


■ **Easy to read Black Line**  
Adopting the latest nanotechnology, "Platinum-Gold Colloid"

■ **Fast test results**  
Reading time is 3-5 min.

■ **Shared use of extraction buffer in Capilia series**  
1 sample can be used for 4 types of tests (Flu, Adeno, hMPV, RSV) at once



more reasonable  
process, less pain  
for patients

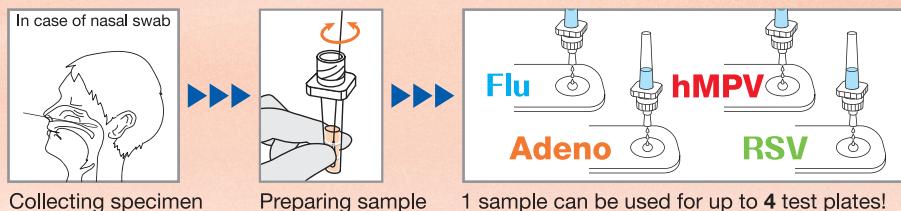

Please follow package insert with each product about shared use of extraction buffer.

# Adenovirus Rapid Test


# Capilia™ Adeno Neo

To detect Adenovirus antigens in pharyngeal swab, nasal swab, nasal aspirate or keratoconjunctivitis swab.





## Test Procedure

### Method of specimen collection

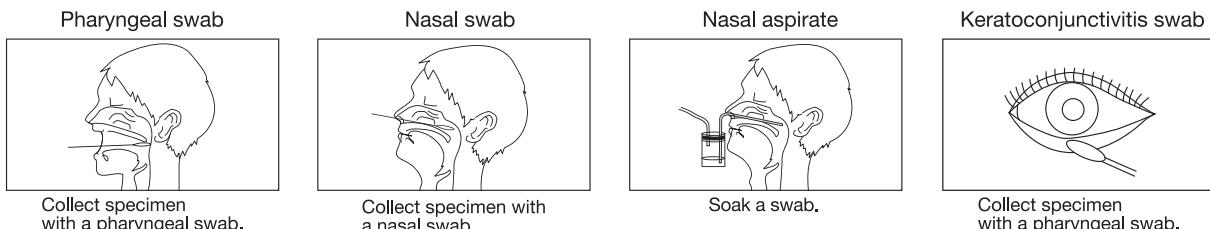

### Sample preparation

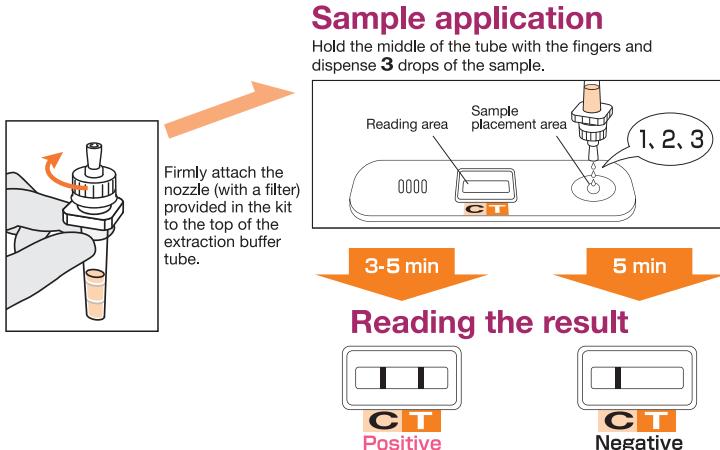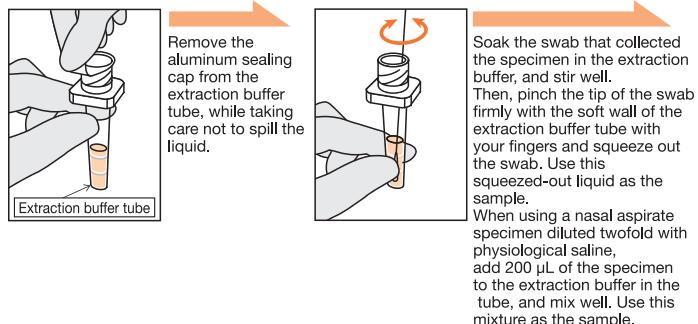

### Reference Data

#### Clinical data

The result of the clinical performance evaluation in Japan  
(Comparison with control products)

Kind of samples	Sensitivity (%)	Specificity (%)	Accuracy (%)	Total number
Pharyngeal swab	100 (56/56)	99.2(125/126)	99.5(181/182)	182

The result of the clinical performance evaluation in Japan  
(Comparison with RCR)

Kind of samples	Sensitivity (%)	Specificity (%)	Accuracy (%)	Total number
Nasal swab	96.5 (55/57)	98.2 (56/57)	97.4(111/114)	114
Nasal aspirate	98.1 (52/53)	98.3 (58/59)	98.2(110/112)	112
Keratoconjunctivitis swab	91.2 (62/68)	98.4(127/129)	95.9(189/197)	197

### Product Summary

#### ■ REF CAAD0370 (20 Tests)

Kit contents: Test Plate×20, Extraction Buffer 0.7 mL×20,  
Nozzle×20

■ Validity: 27 months

■ Storage: Store at 2-30°C

■ Reading time: 3-5 minutes

#### ■ REF CAAD0371 Test Plate (10 Tests)

Kit content: Test Plate×10



761-1, Kamishima, Izunokuni,  
Shizuoka, 410-2325 Japan  
URL : <http://www.tauns.co.jp/english/>