Please read this package insert carefully before use.

English

Capilia™ TB-Neo Extraction Buffer

This product is an extraction buffer using Capilia TB-Neo to prepare a sample that is employed for the identification test of *M.tuberculosis* when using a solid medium for acid-fast bacteria (AFB). Therefore this extraction buffer should not be used for any other identification tests or following methods other than those prescribed. Otherwise, it may affect the test results and accurate results cannot be obtained.

(1) Component

Extraction buffer (Phosphate buffer, detergent, sodium azide (0.09%))

(2) Usage

- 1) Incubate at 37°C for 2 to 4 weeks until the growth of bacterial colonies is confirmed on the solid medium, and then confirm the presence of AFB by acid-fast staining.
- 2) Dispense 0.2 mL of the extraction buffer into a tube.
- 3) Remove 1 µL (equivalent to the amount of a 1 mm-diameter platinum micro-loop) from the bacterial colony that has grown on the solid medium.
- 4) Suspend the collected bacteria in the extraction buffer in the tube.
- 5) Close the tube with a stopper and fully suspend with a vortex mixer. Then, use the bacterial suspension for samples.
- 6) Please follow the use of Capilia TB-Neo specified in the test procedure.

(3) Precautions when using and handling

- 1) This product is a Capilia TB-Neo extraction buffer for a solid medium of Capilia TB-Neo. Please do not use for any other purpose.
- 2) This product should be stored between 2°C and 30°C.
- 3) When dispensing the extraction buffer into a tube from the bottle, please make sure that there is no contamination by tubercle bacilli or any other bacteria.
After use, please close the bottle with a stopper.
- 4) The suspension of the bacteria should be done thoroughly.
- 5) The extraction buffer contains sodium azide. If the solution gets into your eyes or mouth, or comes into contact with your skin, immediately flush with a large quantity of water as an emergency measure. If you still feel some abnormality, see a doctor for treatment.
- 6) When solutions containing sodium azide continue to be discarded over a long period of time, explosive metallic azide may be produced if a drain is made of metal. Therefore, they should be discarded with a large quantity of water.
- 7) Because test tubes, platinum micro-loops, remaining specimens and other test materials/devices used for specimens may cause infections, they should be autoclaved before disposal. When sterilized used reagents or other test materials are disposed of, they should be treated in accordance with the laws and regulations concerning medical waste disposal and water pollution control.
- 8) Do not use this product after the expiration date.

(4) Storage

Storage: Store at 2°C to 30°C

(5) Packaging Unit

[REF] CATB0877 Capilia TB-Neo Extraction Buffer 20mL

(6) References

Abe C, et al.: Simple and rapid identification of the *Mycobacterium tuberculosis* complex by immunochromatographic assay using anti-MPB64 monoclonal antibodies. *J Clin Microbiol.* 1999;37:3693-3697.

INQUIRIES

 TAUNS Laboratories, Inc.
761-1, Kamishima, Izunokuni,
Shizuoka, 410-2325 Japan

FAX : +81-558-76-0022

[ECREP] Emergo Europe
Prinsesegracht 20
2514 AP The Hague
The Netherlands

GLOSSARY OF SYMBOLS

	CE Marking (European directive 98/79/EC on <i>in vitro</i> diagnostic medical devices)
	<i>In vitro</i> diagnostic medical device
	Temperature limitation
	Use by YYYY-MM
	Batch code
	Contents sufficient for <n> tests
	Authorized representative in the European Community
	Manufacturer/Manufactured by
	Consult instructions for use
	Keep away from sunlight

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